## ACTIVE SPORT LIGHT

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**English version** on next page

**Users with visual impairments can find the PDF files together with further information on our website:**

**<[www.viteacare.com](http://www.viteacare.com)>.**

**👉 Contact your specialist dealer when required.**

**Alternatively users with visual impairments can have the documentation read out by a helper.**

# Table of contents

<b>Introduction</b> .....	<b>26</b>
<b>Product description</b> .....	<b>27</b>
<b>Indications</b> .....	<b>28</b>
<b>Contraindications</b> .....	<b>28</b>
<b>Usage</b> .....	<b>28</b>
Unfolding the wheelchair .....	28
Mounting of footrests .....	29
Hip belt installation .....	29
To seat a patient in a wheelchair, you must: .....	30
Driving a wheelchair by a caregiver .....	30
Moving on sloping surfaces .....	31
Driving off thresholds or curbs .....	31
Driving over thresholds or curbs .....	31
Transporting the wheelchair by car .....	32
Mounting or dismounting rear wheels .....	33
Adjustment of the brake mechanism .....	33
Footrest length adjustment .....	33
Inflating tires .....	34
Wheel pressure and braking force .....	35
<b>Safety of use</b> .....	<b>36</b>
<b>Technical specifications</b> .....	<b>39</b>
<b>Service and repair</b> .....	<b>40</b>
<b>Care and maintenance</b> .....	<b>40</b>
Cleaning .....	40
Disinfection .....	40
Maintenance plan .....	41
Product disposal .....	42
<b>Symbols</b> .....	<b>42</b>
<b>Contact details</b> .....	<b>44</b>

## Introduction

Thank you for choosing our product. We firmly believe that it will meet your expectations. Please read this document before using it for the first time. This instructions for use is provided to give you the necessary information for safe use. Correct application of the recommendations will significantly contribute to prolonging the life and aesthetics of the product. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

# ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

### WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Product description

Illustration 1. List of components.

Nr	Item description
1	Handle
2	Backrest
3	Armrest
4	Rear full tire (PU)(Inflatable tyre as an option)
5	Metal handrim
6	Seat frame
7	Rear wheel quick release system
8	Front wheel fork
9	Front wheel
10	Brake lever
11	Footrest frame
12	Footrest handle
13	Cross frame
14	Footrest platform
Option	Optional equipment available as a package: cushion, anti-tip wheels, waist belt, calf belt.

## Indications

The wheelchair is used in order to commute by people with disabilities e.g. after the spinal cord damages, cerebral palsy or various lower limb injuries, neuromuscular diseases, post-traumatic cranial cerebral or for geriatric patients.

It is also intended for people who want to actively and comfortably participate in everyday life.

## Contraindications

Contraindications to a sitting position. Conditions that require stabilization of the upper spine and head. Patients with mental disorders should not be left unattended in the wheelchair.

## Usage

After unpacking the package, check that it contains all parts according to the list of components in illustration 1. Make sure that all parts have no visible mechanical damage. If in doubt, contact your distributor.

Do not use the wheelchair until all its components have been properly connected.

## Unfolding the wheelchair

- Place the wheelchair with the rear wheels facing each other.
- Pull the handles of the wheelchair outward (Illustration 02.).
- Position the wheelchair with the front wheels facing you.
- Press down on the seat frames until the wheelchair is fully unfolded.

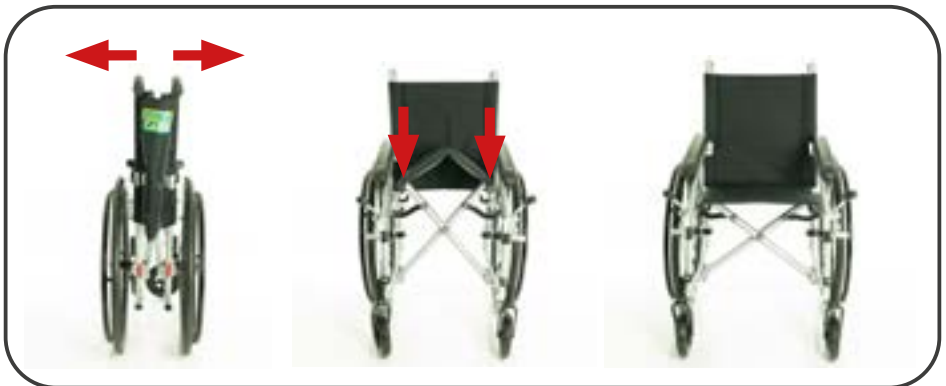
Illustration 2. Unfolding the frame

## Mounting of footrests

- To attach the footrests, hang the footrest mounting brackets on the footrest handles (Illustration 3.) and rotate them as far inward as possible until the footrest locks into place and a distinctive click is heard. (Illustration 4.)

Illustration 3. Footrest handles.

Illustration 4. Prepare the footrest.

- Make sure the footrests are locked in place.
- Then unfold the footrest platforms.

Illustration 5. Footrest lock. Pull upward to unlock the footrest rotation.

Illustration 6. Mounted footrests.

## Hip belt installation

### WARNING!

Check the connections of the wheelchair components regularly and, if in any doubt, contact your distributor. Regularly check the tightness of the hip belt mounting bolt with the armrest.

**Step 1** - Unscrew the armrest mounting screw at the rear backrest (5mm Allen key).

**Step 2** - Using a screw, fit the strap between the armrest handle and the wheelchair frame.

**Step 3** - Insert the screw into the threaded hole in the frame and then tighten firmly.

### To seat a patient in a wheelchair, you must:

- Place the wheelchair in a position that will make it easier to transport the patient (at the bed, chair or within reach of the lift). Lock both wheelchair brakes, fold or remove footrests.
- Place the patient on the wheelchair.
- Mount or unfold the footrests, place the patient's feet on them.
- Place the patient on the wheelchair in an upright sitting position (knees should be bent at  $\sim 90^\circ$ ).
- The height of the armrest position can be changed for the patient's preference.
- To do this, press the armrest button (Illustration 7.) and pull the armrest upward to make the armrest higher, or press the armrest button slightly downward to make the armrest lower.

Illustration 7. Armrest Button.

Illustration 8. Raised armrests.

### Driving a wheelchair by a caregiver

- Always make sure that no part of the patient's body or clothing is at risk of catching on the wheelchair's moving parts before starting the ride.
- Unless the patient can independently maintain a sitting position, you should (additionally) equip the wheelchair with stabilizing belts prevent the patient from slipping off the seat).
- The wheelchair should be operated with both hands.

## **Moving on sloping surfaces**

- When moving on inclined surfaces, use extreme caution and reduce speed.
- Do not move on excessive inclines. The maximum permissible upward and downward slope of the ground is 5% (2,86°).
- Consider using a caregiver, if the person in the wheelchair does not have the strength to control the wheelchair, apply the brakes.
- To prevent tipping over when moving on upward sloping ground, lean forward slightly to shift the center of gravity forward.

## **Driving off thresholds or curbs**

### **Forward without assistance from a caregiver:**

- An experienced wheelchair user can negotiate thresholds or curbs independently.
- You can get off a low curb by moving forward. Make sure that the footrest platforms do not touch the ground.
- Keep your balance on the rear wheels to reduce the pressure on the front wheels.
- Next, you should negotiate the curb.
- Forward with the help of a caregiver:
- Higher curbs can be overcome with the help of a caregiver.
- The caregiver should tilt the wheelchair back slightly.
- The curb should be overcome by moving on the rear wheels.
- The wheelchair must then be supported on all four wheels.

### **Forward with the help of a caregiver:**

- Higher curbs can be negotiated with the help of a caregiver.
- The caregiver should tilt the wheelchair back slightly.
- The curb should be overcome by moving on the rear wheels.
- The wheelchair must then be supported on all four wheels.

### **Backwards without assistance from a caregiver:**

- An experienced wheelchair user can negotiate thresholds or curbs independently.
- Turn the wheelchair so that both rear wheels face the curb.
- Lean forward to shift the center of gravity forward.
- Bring the wheelchair closer to the edge of the curb.
- Then, using the rim in a controlled motion, push the wheelchair off the curb.

## **Driving over thresholds or curbs**

### **Forward with the help of a caregiver:**

- Make sure the footrests do not touch the curb.
- The caregiver should lean the wheelchair back enough to lift the front wheels over the curb.
- The person sitting in the wheelchair should lean backward, transferring the center of gravity over the rear wheels.
- Drive the front wheels to the curb and then drive the rear wheels over the curb.
- Higher curbs can be overcome by driving in reverse with the help of a caregiver.

- Turn the wheelchair so that it faces the curb with the rear wheels.
- The person sitting in the wheelchair should lean backward, transferring the center of gravity over the rear wheels.
- The caregiver should pull the wheelchair to the curb.
- The person sitting in the wheelchair should return to a normal position in the wheelchair.

**Forward without the help of a caregiver:**

- An experienced wheelchair user can negotiate thresholds or curbs independently.
- Drive the front wheels up to the edge of the threshold or curb.
- It is important to make sure that the footrests do not touch the threshold or curb.
- Now you can drive the front wheels to the curb.
- Then drive the rear wheels onto the curb.

**Transporting the wheelchair by car**

- To fold the wheelchair for easy transport, remove the seat cushion.
- Next, fold the footrest platforms and remove the footrests.
- Then pull upward on the seat frame (Illustration 9.), this will fold the wheelchair.
- Remove the wheels only using the rear wheels quick removal system.
- To install or remove the rear wheels, push in the locking mechanism of the wheel coupling axle - quick release (Illustration 10.) and slide out or insert the running wheel.

**ATTENTION!**

Carry the wheelchair by holding only the frame elements. Never hold by the armrests or footrests.

Illustration 9. Folding the wheelchair.

Illustration 10. Attaching or disassembly of wheels.

## Mounting or dismounting rear wheels

- In addition, the wheelchair is equipped with a system of quick release rear wheels.
- To install or remove the rear wheels, push in the quick release mechanism (Illustration 10.) of the wheel and slide out or in the rear wheel.

## Adjustment of the brake mechanism

Illustration 11. Brake lever.

Illustration 12. Brake adjustment.

- To apply the brake, pull the brake lever toward the front wheel until the brake is tight against the rear wheel.
- Make sure that the brake lever does not bounce back on its own when you try to move the wheelchair.
- To disengage the brake, pull the brake lever toward the rear wheel.
- To change the position of the brake mechanism, loosen two screws from the inside of the wheelchair, using a 5mm Allen wrench. (Illustration 12.).
- Move the entire mechanism towards the wheel and tighten the screws.
- After switching the brake back on, the wheel should not rotate.

### SUGGESTIONS!

Test the correctness of the brake adjustment on the wheelchair with the patient.

## Footrest length adjustment

- The footrest is adjustable in length.
- To set the desired length of the footrest, remove the footrests.
- There is a footrest adjustment screw at the bottom of the footrest.
- Use a 6mm allen wrench to loosen the footrest adjustment screw. (Illustration 13.)
- Slide in or out the footrest platform embedded in the footrest frame as required.
- After selecting the correct length of the footrest, tighten the footrest adjustment screw and attach the footrest to the wheelchair.

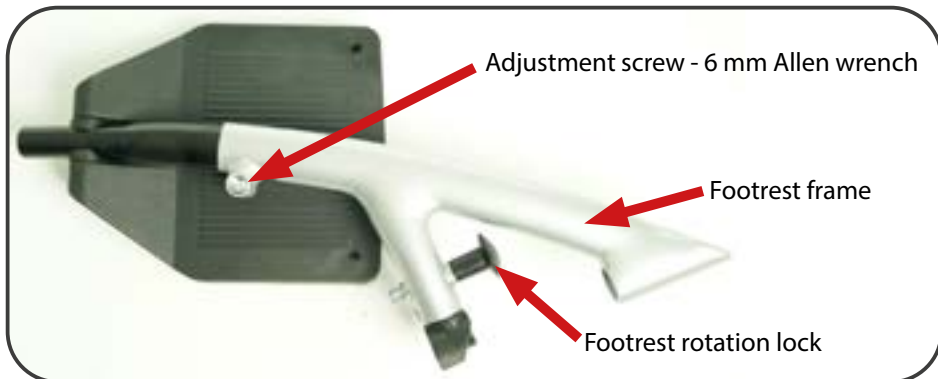
Illustration 13. Footrest length adjustment.

## Inflating tires

*Refers to wheelchairs equipped with inflatable wheels.*

Illustration 14. „Schrader“ valve  
(automotive)

Illustration 15. „Woods“ valve  
(bicycle, Dunlop)

- Before inflating tires, identify the type of valve that is located at the tire. It can be one of the two valves shown in illustration 14. and 15.
- Once you have identified the valve, prepare the pump designed for the type of valve installed on the wheelchair.
- Most new inflators are equipped with both car or bicycle type valve support. Many of them allow you to reverse the tip. The smaller hole facing outward will be for “Woods”(bicycle), the larger for “Schrader”(car). If the pump does not support a bicycle valve, you will need to equip yourself with the appropriate adapter, which is available at any bicycle store.
- Unscrew and remove the plastic cap from the valve and put it in a safe place. It is important not to lose it, as the valve will become dirty, leading to difficulty in pumping and faster loss of air.

- Apply the pump nozzle to the valve and check that it adheres properly. If necessary, it is advisable to press the pump against the valve so that air does not escape to the sides.
- If the pump has a lever with a clamp at the nozzle, clamp it. If the pump nozzle and the valve have a threaded end, they should be screwed into each other. This will ensure that the air will fall straight into the inner tube, and the nozzle will not detach from the valve.
- When proceeding to pumping, periodically check how much pressure is in the tires. The best way to do this is by pressing the tire with your thumbs at the end of the tread. If it is possible to make only a millimeter dent, it means that there is enough air in the tire. If necessary, continue pumping.
- When you have finished pumping, remove or unscrew the nozzle from the valve by pressing its outlet with your thumb. It is important to do this, smoothly enough so that not too much air escapes.
- Screw the plastic nut onto the valve.

## **Wheel pressure and braking force**

- Adjust the rear wheel pressure according to the weight of the user and then, with the assistance of the caregiver, check the operation of the parking brakes (loaded with the weight of the user, the wheelchair facing forward in the direction of the slope, must not roll on a plane with an incline of 15% ~ 9°).
- After all adjustments, remember to carefully check all connections.

### **ATTENTION!**

Observe the correct degree of inflation of the wheels according to the sizes specified by the manufacturer on the tire pages.

Use a pump with a pressure gauge or a bicycle hand pump.

## Safety of use

### WARNING!

Any „serious incident“ related to the product must be reported to the manufacturer and the competent authority of the Member State where the user resides.

### ATTENTION!

The product is intended for disabled people with musculoskeletal disorders. If you are not sure if your health condition allows you to use the product consult a health care professional.

### WARNING!

Do not modify the product in any way; this will void the warranty.

### WARNING!

If the maximum user weight is exceeded, the warranty will be void. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

### WARNING!

Before using the wheelchair, please read these instructions for use and keep them for future reference.

- a. Before using the wheelchair, read this manual and keep it for future reference.
- b. When getting on or off the wheelchair, apply the brakes!
- c. Before sitting down or getting out of the wheelchair, do not stand on the footrest plate, fold it down beforehand or push the footrests aside completely.
- d. It is unacceptable to move the wheelchair with the user.
- e. When moving the wheelchair through doorways and other passageways, make sure there is enough space on both sides to avoid injuring your hands or damaging the wheelchair.
- f. When using public roads, follow traffic regulations.
- g. As with other vehicles, do not drive the wheelchair under the influence of alcohol or other drugs or intoxicants. This also applies to indoor driving.
- h. The wheelchair is not designed to transport the patient up stairs, as a stretcher or by pulling the patient on it. The wheelchair's design and connecting parts are not designed for such heavy loads if it is used in this way. This risks weakening or damaging the entire support structure of the wheelchair and a possible accident!
- i. The wheelchair is designed for indoor and outdoor use only on paved and level surfaces.
- j. Use of the product on a surface that does not provide secure support for the user, i.e. icy, wet, greasy, heavily dusty is not allowed. - risk of falling.

- k. The wheelchair can only be used on surfaces where all four wheels of the wheelchair and the two front wheels of the extension touch the ground and the contact is sufficient to drive the wheels adequately.
- l. A caregiver's assistance is required to overcome difficult architectural barriers (steep or long ramps, high steps, etc.).
- m. If the user does not have sufficient experience in using the wheelchair, ask the caregiver to assist in overcoming obstacles.
- n. Do not leave the wheelchair with the user in the immediate vicinity of an open flame source.
- o. Be aware that the surface temperature of the upholstery and frame components may increase after exposure to the sun or other heat source, while in cold temperatures there is a danger of very high cooling of the wheelchair's metal parts. This can cause burns or frostbite to body parts.
- p. Move the wheelchair by grasping the frame structure. Never carry the wheelchair by the armrests or by the footrests.
- q. The wheelchair must not be used for use as a seat in a motor vehicle (bus, train).
- r. The wheelchair should always be able to be folded and unfolded easily. If you have problems with folding, read the instructions again, or contact your distributor.
- s. When moving on sloping surfaces, special care should be taken. The maximum permissible angles of upward and downward inclination of the ground are 5%.
- t. Brakes are not used to slow down the wheelchair during movement, they should be used only to prevent, unwanted movement of the wheelchair.
- u. Both brakes should be properly adjusted. Incorrect adjustment of one of the brakes, resulting in the locking of only one rear wheel, which on steep inclines can cause the wheelchair to rotate around the braked wheel, which is difficult to control.
- v. Tire pressure should be checked regularly. If the tire pressure is too low than the recommended pressure, the effectiveness of the brakes can be significantly reduced!
- w. When picking up objects from the floor, do not reach backward, as you risk tipping backward.
- x. When reaching objects, avoid excessive leaning out of the wheelchair as there is a risk of falling.
- y. Do not reach objects from the floor in front if this requires you to slide out of the wheelchair.
- z. Any activity such as getting in, getting out of the wheelchair, changing the patient's position, or stopping the wheelchair requires locking the running wheel brakes.
- aa. Do not leave the patient in the wheelchair (even with the brakes applied) on descents, ramps, mezzanines, etc.
- ab. Each time after mounting the wheels, check that the wheel coupling axle lock has worked, when mounting, the characteristic click of the latch should be heard, in addition, the wheel should be strongly jerked without pressing.
- ac. Using the parking brake while driving can risk an accident with all the consequences.
- ad. Always after each adjustment, check the correct operation of the brake.
- ae. If the wheel tires are contaminated by oil, grease, water or other agents, braking efficiency may be reduced.

- af. After the wheelchair has been in use for a long time, the braking force may be weakened due to the formation of play between the brake pressure pin and the wheel tire. It is possible to adjust the parking brakes yourself. To do this, loosen the screws locking the position of the brake mechanism, move the entire mechanism towards the wheel to such a position that when the brake is applied again, the wheel does not rotate (test the correctness of the adjustment on the wheelchair with the patient). After completing the adjustment, again secure the position of the mechanism with the locking screws.
- ag. To maintain the performance of the wheelchair, it is extremely important to keep all its components clean, and how it is stored. Storage of the wheelchair in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene will cause a rapid loss of functional and aesthetic parameters of the wheelchair.
- ah. It is unacceptable to use the wheelchair if any visible or perceptible damage has been found, or the degree of wear and tear of the wheelchair's parts indicates a potential risk of an accident (e.g. wear and tear of the running wheels, cracks in the upholstery, etc.). All repairs should be entrusted to qualified personnel, and during the warranty period only to the manufacturer's service department.
- ai. For wheelchairs equipped with inflatable wheels, do not exceed the maximum allowable wheel inflation pressure, the value indicated on the wheel tire.
- aj. When transporting (for example, in the trunk), do not load the wheelchair with anything, it may affect its structure.
- ak. Keep away from children under the age of five.
- al. Be very careful when folding the wheelchair, failure to pay attention can risk getting your fingers caught in the folding parts, within:
- attaching and removing the footrest,
  - when unfolding the wheelchair within the moving parts,
  - In the area of brakes.

**WARNING!**  
Maximum user weight is 115 kg.

## Technical specifications

Maximum user weight	115 kg			
Height of wheelchair / height of handles	915 mm			
Wheelchair width (depending on the model)	610 mm	630 mm	660 mm	690 mm
Wheelchair length	960 mm			
Seat width (depending on the model)	380 mm (15")	400 mm (16")	430 mm (17")	460 mm (18")
Seat depth	395 mm			
Backrest width (depending on the model)	380 mm	400 mm	430 mm	460 mm
Backrest height	420 mm			
Dimensions of the folded wheelchair	915 x 315 x 960 mm			
Weight of wheelchair with all components	11,8 kg			
Weight of wheelchair without components	7,5 kg			
Materials	Aluminum frame, upholstery: polyurethane, polyethylene, rubber			
The size of the rear wheels equipped with a quick disassembly system (quick release), PU tires	Diameter 610 mm (24")			
Diameter of full front wheels	150 mm (6")			
Maximum safe inclination	5% (2,86°)			
Description of the environment in which the product will be used				
Storage and usage temperatures	Od +5°C do +40°C			
Humidity for storage and use	Od 30% do 70 %			
Tools needed for adjustment	Allen wrench: 3 mm, 6 mm			

## Service and repair

Inspection should be performed regularly, and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, contact your distributor.

## Care and maintenance

### Cleaning

- Upholstery and frame should be cleaned with a damp cloth without detergents and other similar agents.
- Remove dirt on a regular basis, e.g.: threads hair, etc. from the wheelchair's moving parts (wheels, footrests).
- Regularly remove dirt from textile elements.
- To maintain the performance of the product, it is extremely important to keep all its elements clean, and the way of storage. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with lack of hygiene will cause rapid loss of functional and aesthetic parameters.
- Do not expose the wheelchair to extreme weather conditions (rain, snow, very strong sun), as certain parts may change color.

### It is not permitted to use:

- solvents,
- toilet cleaners,
- agents with chlorine content,
- sharp brushes and hard objects,
- caustic cleaners.

*Do not clean the wheelchair with pressure or steam units!*

### Disinfection

If the product is used by many people (for example, in a social care home), use commercially available disinfectants. Clean the top surfaces before disinfection. Only tested and approved disinfectants must be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods can be obtained from the national health authority.

#### WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

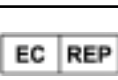
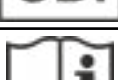
## Maintenance plan

Frequency	Subject of the inspection	Comments
Before the first installation	Inspection of the technical condition of all components in the package.	Check that no components have visible mechanical damage.
Before each start using	Wheel and brake inspection.	Brakes can wear out, check the condition of the brakes before each use. Check the wear condition of the wheels and clean them from dust and sand.
Every two weeks	Attaching bolts, nuts and proper fastening.	Make sure all bolts and nuts are tightened securely.
Every two months	Cleanliness, general condition.	In the case of resulting mechanical damage, contact the distributor immediately.
Once a month	Wheel axles.	Wheel axles should be maintained by spraying a few drops of preservative-lubricant between the axle and the hub, in addition, the wheelchair should be raised so that the wheel under test is above the ground, the wheel should freely turn on the driving axle, and the front wheels also on the directional axis.
Once a month	Condition of quick-connect, threaded and soldered joints.	Attention should be paid to any cracks in the frame material at the soldering points and at the locking holes of the bayonet connections).
Once a month	Tire condition.	Make sure that, the tread of the tires is not worn out, and that the tires have no visible damage.

## Product disposal

The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

## Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)
	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning

	Possibility of jammed fingers
	Do not stand on the footrest
	Permitted temperature
	Protect from moisture
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

## Contact details

### Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl)

[www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.