INSTRUCTIONS FOR USE

Aluminium 4-wheeled rollator with knee support

Product code: Knee Walker

1. Characteristics

4-wheeled rollator with knee support is made of aluminium frame which is coated with paint with metallic lustre. The rollator is equipped with detachable bag, footrest and knee support with adjustable height. The product also features height adjustable handlebar with anatomic handgrips and handbrakes (back wheels), wheels with narrow cross section, 8" in diameter.

The set includes:

foldable frame, knee support, universal footrest, footrest cap, bag with reflective strips, short locking knobs – 3 pcs., long locking knob – 1 pc.

Elements of the rollator:

2. Application

The rollator is designed for patients with reduced mobility who need stable support. It is also recommended to the elderly and people with lower limbs injuries as an aid in keeping balance, improving moving comfort and ensuring safety from falling. The use of the rollator has a positive effect on the return to activity of people after injuries. It guarantees better stabilization, support and security while moving.

NOTE! Rollator should not be used by people with issues or injuries of the knee joint because of high pressure put on that area during usage of the rollator.

3. Assembly

Unfolding and folding the rollator

Picture 1

Picture 2

- Unfold the rollator, fix part A into part B, till you hear clicking sound of a locking pin. Secure the frame with locking knobs.
NOTE! Length between part A and B is adjustable. To adjust it, unscrew the locking knob, taking it out, and then press the pins on both sides, adjusting the frame to an appropriate length. Secure with locking knob afterwards.
- Secure back wheels by putting the long locking knob into the opening located at the back of part B. Correctly mounted locking knob should go through 4 openings of part B (pic. 2). Tighten the locking knob.
- Push the bar forward to connect the handlebar and rollator base (horizontal position). Tighten the safety buckle (see pics. 3-5).

Picture 3

Picture 4

Picture 5

- Secure the handlebar by pulling it up to appropriate length. Put the locking screw through the opening located at the front and tighten it using a knob on the other side.
- Fix the knee support into opening in part B. Pick appropriate height and secure it with locking knob.
- Secure footrest with the cap included.

Picture 6

- Slide the bag into top openings on a special plate located at the front of the bar. Push the bag downwards to secure it in place (pic. 6). For extra security, fix the velcro strap located at the back.
NOTE! Additional insert made of stiff material is located at the bottom of the bag. Use velcro straps to secure it in place.
- To fold the rollator, repeat mentioned above actions in a reversed order. Make sure to: take out all locking knobs with screws, detach part A from part B, take off the knee support and fold handlebar to horizontal position.

Note! While folding handlebar, make sure to unlock the safety buckle and pull to the side to separate bar base from frame.

Brakes

- To stop the rollator, press both of the brake handles at the same time.

Height adjustment of handles

- To adjust the height of the handles, unscrew the locking knobs which are located on the outer side of the frame, select appropriate height and tighten the locking knobs.

4. Technical data

Overall width	44 cm
Knee support dimensions	35 x 18 cm
Overall length	84-89 cm
Seat height from the ground	45,5-58 cm
Handles height adjustment	89-119 cm
Weight	9,6 kg
Maximum capacity	136 kg
Frame color	black

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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