KONSERWACJA:

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MAINTENANCE:

Remove metal splint with hinge prior washing. Padding elements and adjustment bands should be washed up by hand using mild detergents. Do not chlorinate, do not dry clean, wring gently before drying. Do not iron. Do not dry close to the artificial warmth source. Keep away from fire source and do not expose direct sunlight.

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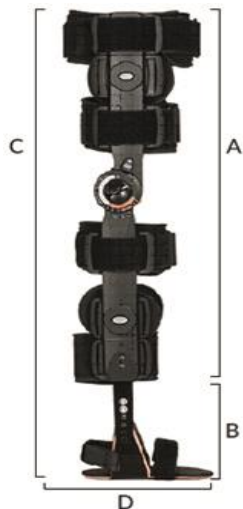
**ATTENTION!**

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

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THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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Universal size

Length: 56 - 72 cm (A)
Height of the foot locking part: 13,5 - 16 cm (B)
The length of the orthosis with the foot locking part: 69,5-88 cm (C)

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INSTRUCTIONS FOR USE

KAFO – Knee ankle foot orthoses

ATTENTION! The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctors or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

1. Adjust the length of the side rails on the pressing part of the femur and tibia (please see the figure), by preloading them to the desired length.
2. Required by therapy limitation of angular range of motion should be chosen and set before putting on the brace.

INDICATIONS:

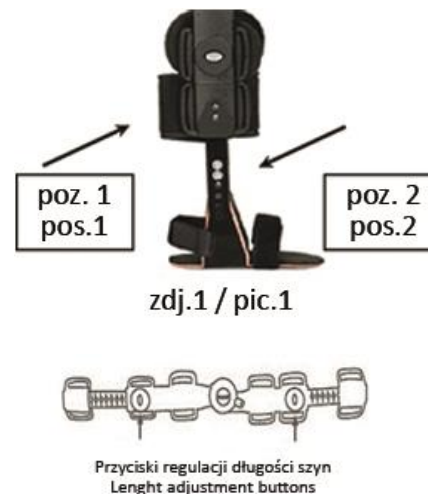
- Conditions which require immobilization or control of knee joint range of motion after surgeries or injuries with necessity of ankle joint immobilisation for better stabilization effect.
- Trauma of knee joint with neurological or orthopaedic syndromes which require immobilization or control knee joint control range of motion combined with paresis feet.

CONTRAINDICATIONS:

- Conditions which require complete immobilization of lower limb intraarticular fractures, fractures of tibia, fibula and femur.

USE:

1. Make sure to select the correct size of the foot immobilizing part. Insert the locking element rate according to the following guidelines:
 - Screw/unscrew the two screws that secure locking element which immobilize feet (pic.1 pos.1) to pelottes knee joint of each site. Remove / put on foot stabilizer

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2. After adjusting all parameters put on the brace – remember to the position the hinge properly (the hinge axis should be little above articular space). The splints should be placed symmetrically on both sides of lower limb. During usage it is important to wear proper type of shoes like: (sports shoes, flat heel, preferably laced). If it is necessary insert foot immobilizing element in the shoe than slide the foot into the shoe, and then wear the orthosis according to the rules described below. Please remember that foot locking element could not be worn directly on the body - should protect foot from irritation by cotton sock.
3. After putting the brace on right place, fasten stabilizing bands starting from those close to the knee joint.